JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com

Transportation

It can transportation by car, and other public transport.

Installation and Using Method

a. Take out the Commde chair from the carton and open the commode chair, Please refer to illustration 1&2

- Please turn the front armrest frame to open,
- Then turn the behind armrest frame, make armrest frames in one level.
- Pick up the cross tube, make sure the U shape of cross tube insert into the seat frame.

illustration 1

illustration 2

b. Put the bucket in, and make sure the shape of seat plate and cover plate insert into the seat frame. Please refer to illustration 3

c. Make sure the foot tube adjustable at the same height, before Using

d. Maximum loading capacity 100kgs. Users are not allowed to exceed the max. loading capacity

Training

When first use commode chair, must have people accompany training how to use, Please make the tubes in the same height and button out, before using. Must use be flat place.

Disposal

JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com


Commde Chairs

Purpose of the device

The purpose of the commde chair help the people with disabilities to toilet, make the people in need by themselves.

Descriptions: Steel Commode chair

Model No.:JL894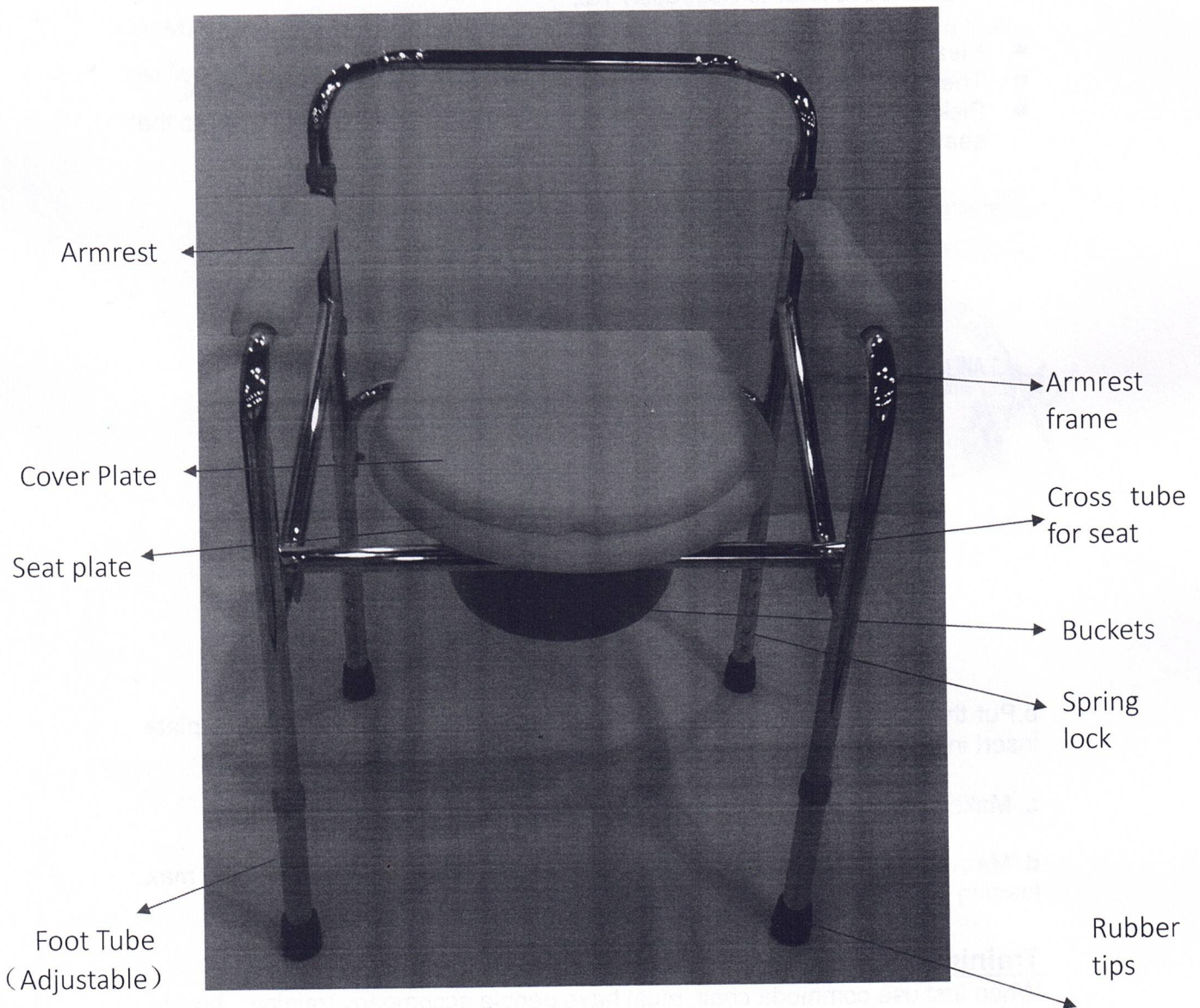

JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com

Put out cover plate, seat plate and bucket. Let the cross tube down ,
and then turn the armrest frames, one to the side, one to back.
Please refer to illustration3& 4

U shape part
of seat plate &
cover plate

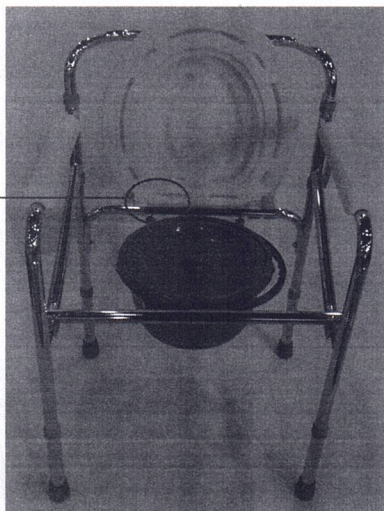
illustration 3

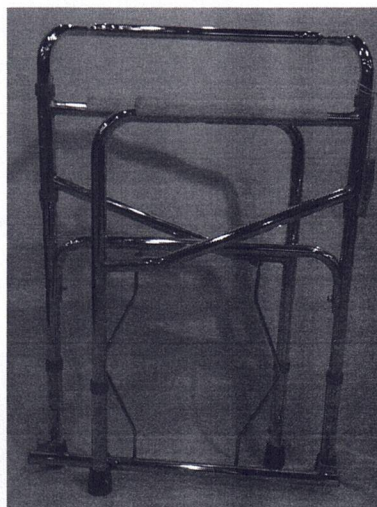
illustration 4

Storage

Storing the commode chair at a dry and ventilated environment ,
Storage Temp about 5℃ to 55℃

Maintenance

- Check the tightness of the screws and spring lock, the stabilization of the walker before using.
- Regular cleaning will prevent the dust of tubes or reveal loose and enhance the smooth operation of your commode chair.

WARNING

ALL warnings and cautions in this manual must be followed.
DO NOT attempt to move up or down an incline with a water, ice or oil film.
DO NOT stand on the frame of the commode chair.

Manufacturer: Jianlian Homecare Products Co. Limited
Dali Xiebian Industrial Park, Nanhai District, Foshan, Guangdong, China

EU Rep: Lotus Global Co., Ltd 
Address: Koningin Julianaplein 10, le Verd, 2595AA, The Hague, Netherlands

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Producător: Jianlian Homecare Products Co. Limited
Reprezentant UE: Lotus Global Co., Ltd