JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com

Instruction of Commode Chair

Purpose of the device

The purpose of the commode chair is to help the people to toilet, make the people in need by themselves.

Descriptions: Steel Commode chair

Model No.:JL897

JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com

Transportation

It can transportation by car, and other public transport.

Installation and Using Method

1. Take the commode out of carton, please refer to the Illustration (1)
2. Open the brace, please refer to the Illustration (2)
3. Fix the brace by the locking pin, assemble the bucket in the seat please refer to illustration (3), (4), (5)

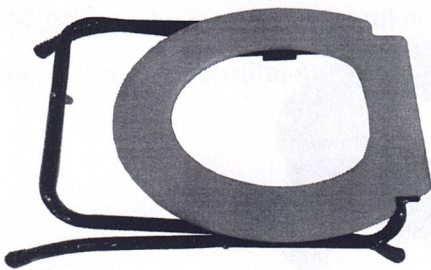
Illustration 1

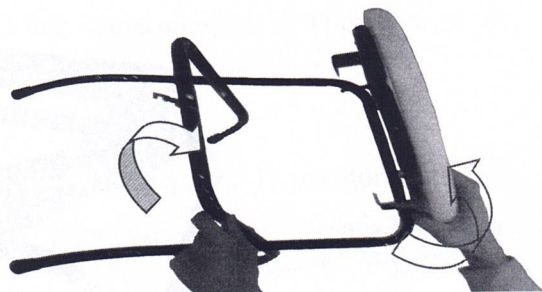
Illustration 2

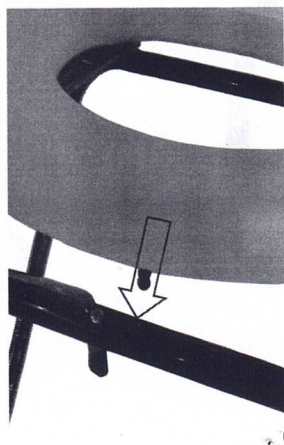
Illustration 3

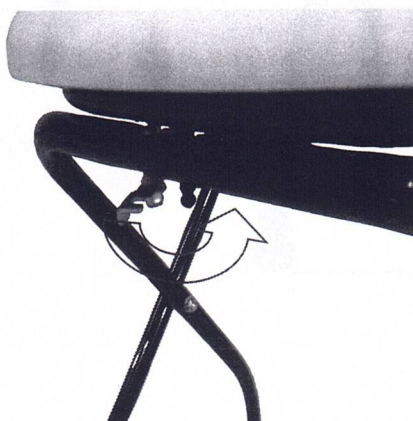
Illustration 4

Illustration 5

JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com

Training

When first use commode chair, please make sure there is somebody who help the user on how to use the commode. Please use the commode on flat place.

Disposal

Unlock the pin and fold the brace. Please refer to the illustration (6), (7) and (8)

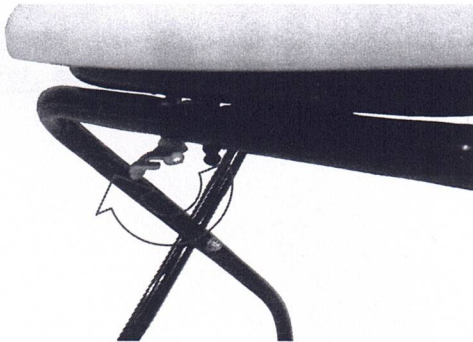
Illustration6

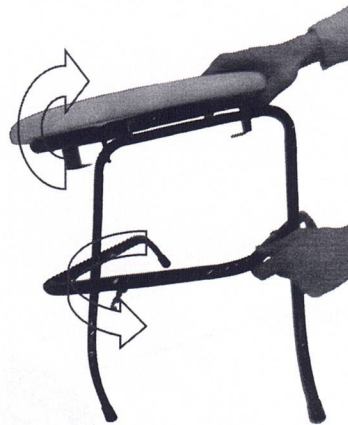
Illustration 7

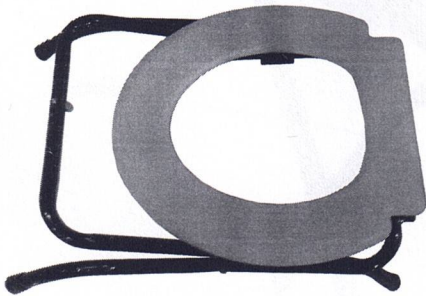
Illustration 8

Storage

Storing the commode chair at a dry and ventilated environment ,
Storage Temp about 5℃ to 55℃

Maintenance

- a. Check the tightness of the screws and spring lock, the stabilization of the walker before using.
- b. Regular cleaning will prevent the dust of tubes or reveal loose and enhance

JIANLIAN HOMECARE PRODUCTS CO. LTD.

Add: Xiebian Industrial Park, Dali, NanHai District, Foshan, Guangdong, P.R.China
Tel: 0086 757 83668093 Fax: 0086 757 83668083 Email:sales@gdjianlian.com

the smooth operation of your commode chair.

WARNING

ALL warnings and cautions in this manual must be followed.
DO NOT attempt to move up or down an incline with a water, ice or oil film.
DO NOT stand on the frame of the commode chair.

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