

JL899 Plastic Armrests & Backrest Commode Chair

Description

- » Durable powder coated steel frame ,can be foldabled easilv.
- » Removable plastic commode pail
- » With fixed armrests and backrest
- » Each leg has a anti-slip rubber tip
- » The chair is foldable for easy storage and transfer



Figure 1



Figure 2



Figure 3

- 1) Take out the product from the box
- 2) Open the seat and lock the two safety buckles
- 3) Insert the bucket as figure 3
- 4) Check the tightness of the screws and the stability of the frame before use;
- 5)Frequent cleaning and regular disinfection; placed in a dry and ventilated environment.

JL899 Scaun de toaleta pliabil cu spatar

Descriere

- » Cadru durabil din oțel acoperit cu pulbere, poate fi pliat cu ușurință.
- » găleată din plastic detașabilă
- » Cu cotiere si spatar fixe
- » Fiecare picior are un vârf de cauciuc anti-alunecare
- » Scaunul este pliabil pentru depozitare si transfer usor

1) Scoateți produsul din cutie

2) Deschideți scaunul și blocați cele două catarama de siguranță

3) Introduceți găleata ca în figura 3

4) Verificați strângerea șuruburilor și stabilitatea cadrului înainte de utilizare;

5) Curățare frecventă și dezinfectare regulată; plasat într-un mediu uscat și ventilat.